


How to Use **D-HELP**[®] The Defogging Heated Cleaning System

PREPARING D-HELP

1. ACTIVATE DEVICE

- Press black button to Activate
- Activate D-HELP at least 2 minutes before use
- Make sure Blue Light is on (Light indicates heating. Light may be difficult to see before room is darkened.)

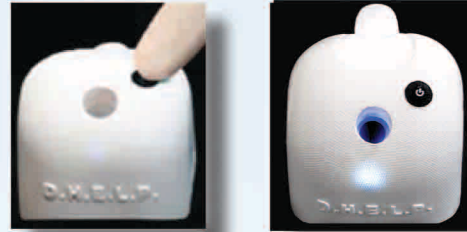

2. INSERT SCOPE INTO D-HELP

- Push scope in firmly and rest scope on back table
 - It is OK if some fluid spills from D-HELP while scope lays sideways
 - There is no risk of overheating from the light-source while scope is inside D-HELP
 - Use the attached rubber reducer for 5mm scopes



3. WHITE BALANCE SCOPE

- Activate White Balance anytime while scope is inside D-HELP
 - If light intensity is too high, white balance may fail. Pull scope away from target or reduce light intensity. If necessary, white balance with the Micro-Pad. DO NOT use Gauze.

4. ATTACH DEVICE TO DRAPES

- Secure to surgical field by peeling the adhesive bottom or secure with a clamp
- Do not hold scope upright. The scope can rest flat on the field with the adhesive flap



Placement Option 2



You do not need to place the light on standby when scope is inside D-HELP.

Placement Option



USING D-HELP

Follow these simple steps before the scope is first inserted and anytime the scope is removed from body.

- **INSERT** into D-HELP for at least 5 seconds.

This step is important. Quickly inserting the scope in and out is insufficient to warm the scope or break up debris from the lens.

- **CLEAN** the lens using a MicroPad or on D-HELP's surface. Do not use gauze to clean lens.

Under hi-fog conditions, do not wipe away all of the anti-fog liquid from the lens. Tap or shake the scope to remove any excess fluid without wiping away all of the Anti-fog coating.

- **RETURN** scope quickly from D-HELP into the body to prevent the scope from cooling.

USING The TrocarWipe

- Use the small foam head for 3-7mm trocars, and the large foam head for >7mm trocars.
- Grasp the blue handle and push the foam head straight into the trocar. Do not release the TrocarWipe and do not bend.