

New Wave Surgical

LAPAROSCOPIC
CARE KIT WITH
D-HELP® FROM
NEW WAVE
SURGICAL >>>>




THE **ONLY SYSTEM** DESIGNED TO KEEP THE LAPAROSCOPE LENS DEFOGGED AND CLEAN FROM START TO CLOSE.

www.newwavesurgical.com

BEFORE THE CASE

At least 5 minutes prior to the case, activate D-HELP®. The anti-fogging solution warms above body temperature preventing fogging of the scope lens during initial insertion into the abdomen. The design of D-HELP will **maintain the warm anti-fogging fluid for up to 5 hours**. This Laparoscopic Care Kit allows surgeons to perform cases more efficiently.

THREE STEPS TO PREVENT LENS FOGGING AT THE START OF THE CASE

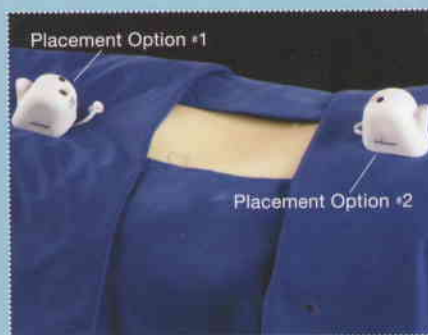


1. Activate D-HELP, press the black button on the top of the unit **at least 5 minutes prior to trocar insertion** which allows the anti-fogging fluid to heat up inside the unit. A blue light will illuminate once the device has been activated. ●●●●

2. **Insert the laparoscope into D-HELP firmly** and rest the scope on the back table, which protects the lens. ●●●●

3. After the patient is draped, bring the scope and D-HELP up to the operative field. Immediately before the trocar is inserted, attach the camera and light source to the scope and **easily perform the white balance**. ●●●●

IMPORTANT: For best results, the laparoscope should go directly from D-HELP into the abdomen.

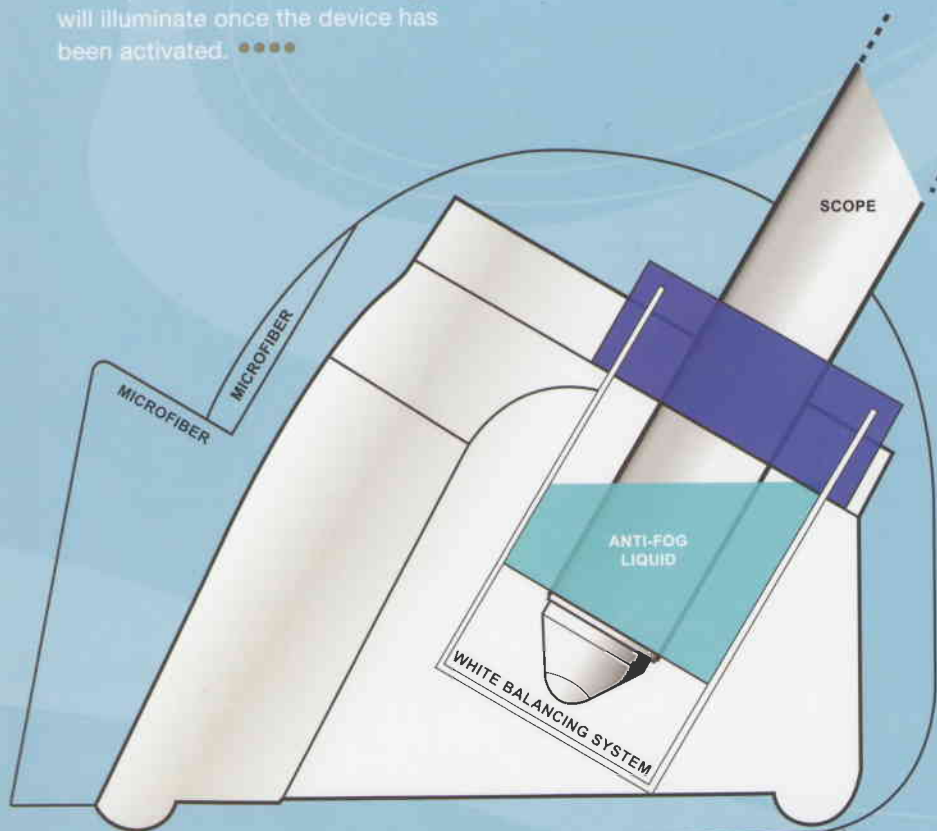

DURING THE CASE

For **best visualization**, it is important to keep the laparoscope lens clean at all times during the procedure. For easy access to D-HELP®, affix the unit to the patient drapes by peeling away the adhesive backing on the flap. The bottom hinge allows the scope to rest flat on the drapes when it is not in use and **avoids risk of overheating from the light source**. For added stability, use a clamp.

Challenges To Maintaining Good Visualization During the Case include:

Laparoscopic Care Kit Components

New Wave Surgical Laparoscopic Care Kit with D-HELP **overcomes visual challenges** associated with laparoscopic procedures by doing the following:

Body fluids and tissue collect on the laparoscopic lens.



D-HELP's warm anti-fogging solution breaks down fats as well as washes fluids and tissues from the lens and may be used as many times as needed during the case. Insert the scope for **AT LEAST 5 seconds** to allow body fluids and tissues to be released from the camera lens. <<<<

Remnants of tissue remain on the laparoscopic lens.



Due to its microfiber surface, D-HELP **enables surgeons to clean the scope with one hand** while the other hand remains free. <<<<

Remnants of tissue remain on the laparoscopic lens that are not able to be removed by other methods.



The **MicroPad**®, unlike surgical gauze, **does not leave lint, dust or smudge tissue oils on the lens**. With **more than 200,000 strands per square inch**; the MicroPad is extremely soft and **does not scratch the laparoscopic lens**. <<<<

Due to Positive pressure, inside the insufflated abdomen, **body fluids can collect inside the sleeve of the trocar port**. This can soil the laparoscope lens during reinsertion.



The **TrocarWipe**® is designed to clean the fluid and/or tissue inside the sleeve of the trocar. Because it has 2 sizes (5mm and 10/12mm), the TrocarWipe cleans ALL trocar sleeves. **This prevents the clean camera lens from being compromised**. <<<<

LAPAROSCOPIC CARE KIT WITH D-HELP® REDUCES TOTAL OPERATING ROOM COSTS

New Wave Surgical Laparoscopic Care Kit with D-HELP is beneficial to providing surgeon's the tools needed to keep the laparoscope lens clean from the start of a case to close. **In addition, the system is competitively priced and reduces overall operating room costs.** Contact a New Wave Surgical representative today.

1. **Replaces ALL other defogging products.**
2. **Saves time** in the operating room.
3. This system **protects the scope lens**, which leads to **lowering the maintenance and repair costs for laparoscopes.**

ITEMS INCLUDED IN THE NEW WAVE SURGICAL LAPAROSCOPIC CARE KIT WITH D-HELP:

1. **(1) D-HELP unit with reducer** which is **compatible with ALL laparoscopes** (from 5mm-12mm)
2. **(2) 6" x 8" MicroPads***—which are x-ray detectable
3. **(1) TrocarWipe***—which is x-ray detectable with a 5mm and 10/12mm cleaning tip

D-HELP is covered by the following U.S. Patent US Patent No. 7,080641B2 and US Patent No. 7,311,660B2 other patents pending, and corresponding foreign equivalents. "New Wave Surgical", the "New Wave Surgical Logo", "MicroPad", "DHELP", "Microfiber Surgical Sponge", "Intraprocedure endoscopic care" and "TrueWhite", are all trademarks of New Wave Surgical Corporation. Copyright © 2006

© 2010 New Wave Surgical Corporation | www.newwavesurgical.com

Designed by: © K2M Designs | www.k2mdesigns.com

Product Photography: © Tyllie Barbosa | www.tylliebarbosa.com



For a Product Evaluation, Contact:

P: 866.346.8883

F: 866.586.6793

E: sales@newwavesurgical.com

www.newwavesurgical.com

New Wave Surgical Corporation
3700 NW 124th Avenue | Suite 135
Coral Springs, Florida 33065

Reference Code

Product Code: #21-345

Rules and Restrictions

Minimum Order = **1 case**
(includes 10 sterile kits/case)

Shipping

New Wave Surgical (NWS) ships ground UPS at no additional charge. **Please allow up to 5 business days for delivery.** Expedited shipping requests can be scheduled for an additional charge. **Orders received after 3 p.m. EST will be scheduled for shipment on the next business day.**